snuza[®] **HERO^{SE}**

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Thank you
for choosing
snuza[®]

and for letting us be a part of your
parenting journey.

We would love to see a photo of your little one
wearing their Snuza monitor!

Please tag us on social media:

@snuzamonitors

@SnuzaUSA

We'd greatly appreciate your review on our
social media platforms, Amazon or via
support@snuza.com.

Your feedback helps to further our work in
promoting safe sleep for infants worldwide.

IMPORTANT: Register your Snuza monitor to
ensure that it is not a pre-owned device. Follow the
instructions in the Warranty section of this booklet.

The Snuza Family
www.snuza.com

30 Days extended warranty

IMPORTANT: proof of purchase is required to validate warranty.

This monitor comes with a 1-year warranty that starts from the date of original purchase, verified by a sales receipt. We'll extend your warranty by 30 days when you register your device within 7 days of purchase.

Here's how to extend your warranty:

1. A. Register as a user on snuza.com/register, follow the prompts to complete registration and add your device, **OR**
B. If you have already registered, log in and scan the QR code on this card to register your device, or navigate to www.snuza.com/account/device/add/
2. Enter your device's serial number and your child's details
3. Upload your proof of purchase and enter the purchase date, and save

To get product support:

1. Log into snuza.com
2. Go to your product's dashboard and click on your device's serial number
3. Click on the 'Get Support' button
4. Enter your product support query, then click the 'Send' button
5. We will be in contact via email to try and resolve your issue
6. If we are unable to resolve the issue, then the warranty claim process becomes available
7. You can do this either on your product's dashboard by clicking the 'Report fault' button or the link provided in the email that you will receive

If you have any feedback or questions, please contact us via your product's dashboard by clicking the 'Send a message' button or email us at support@snuza.com.

Welcome to the Snuza Family!
The Snuza Team

Hero^{SE} Quick Start Guide

Scan the QR code for more detailed information

For support contact us through support@snuza.com

1. Prepare

- 1.1 Switch on by pressing the 'ON/OFF' button.
- 1.2 A start-up test will illuminate the Indicator lights, vibrate the stimulator, play the start-up tone and perform a battery test. Your Hero^{SE} is now ready for use.
- 1.3 **IMPORTANT:** Handle the Hero^{SE} with care to prevent damage to the clip or sensor. Avoid bending the silicone Movement Sensor.

2. Attach

- 2.1 Fold over the top edge of the nappy/diaper.
- 2.2 Attach the Hero^{SE} squarely onto the folded edge of the nappy/diaper.
- 2.3 Ensure the Hero^{SE} fits snugly and the silicone Movement Sensor is in contact with the baby's tummy.
- 2.4 **IMPORTANT:** Ensure no skin pinching by gently sweeping your fingers between the skin and clip.

3. Monitor

- 3.1 When abdominal movement is detected, the Movement Indicator Light flashes with each abdominal movement.
- 3.2 If no abdominal movement is detected for 15 seconds, the Hero^{SE} vibrates and beeps with a red flashing Movement Indicator Light.
- 3.3 If abdominal movement is detected within 5 seconds after a vibration, the Movement Indicator Light flashes red on each subsequent abdominal movement to indicate that there was a vibration event.

- 3.4 After three vibrations, the Rouse Warning is activated (red flash and beeps every 2 seconds). This feature can be deactivated. See section 8.2 of the user manual.
- 3.5 If the period of no abdominal movement reaches 20 seconds, the full audible alarm will sound alerting the parent/caregiver to quickly intervene.
- 3.6 If there are less than 8 abdominal movements per minute, the Movement -Rate warning is activated - Movement Indicator Light flashes yellow with a simultaneous dual-tone audible beep every second. This feature can be deactivated. See section 8.3 of the user manual.
- 3.7 To turn off, press the 'ON/OFF' and 'MODE/OFF' buttons simultaneously.
-

Precautions:

- To avoid pinching the baby's skin, gently sweep your fingers between the clip and skin after attachment.
 - Only use the Hero^{SE} when the baby is in a motionless bed/cot/crib.
 - Not suitable for co-sleeping or bed sharing.
 - Movement from external sources such as being pushed in a pram/stroller or travelling in a vehicle or aircraft may be detected by Hero^{SE}'s sensor.
 - Hero^{SE} should not be relied upon where external sources of movement are present.
 - This movement monitor may not be able to detect all life threatening events.
-

Disclaimer: Customers use this monitor at their own risk. Snuza International (Pty) Ltd, its members, employees, agents, contractors, representatives, manufacturers, distributors and associates ("SNUZA") shall not be held liable for any loss, damage or injury suffered by the customer or any other person arising from, or in connection with, the use or manufacture of the monitor, including loss, damage or injury suffered by the customer or any other person as a result of any defects in respect of the monitor, and all warranties, and any other liability from whatsoever cause arising, are hereby excluded. The customer hereby indemnifies Snuza International (Pty) Ltd in respect of any such liability. **Guarantee:** Snuza International (Pty) Ltd guarantees this product to be free from defects in materials and workmanship for a period of 12 months from the original date of purchase, verified by a sales receipt.

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