08/2014

Portable Baby Movement Monitor

Instruction Manual

READ INSTRUCTIONS IN FULL BEFORE USE

Important: Snuza Go! is a baby-care aid, not a medical device. It is not intended for the diagnosis, prevention or monitoring of a disease, injury or handicap, including SIDS.

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PGM02101

ENGLISH

SNUZA is a registered trademark of
Biosentronics 

Made in South Africa
Patented

Patent numbers:

2008212520 | 2,677,990 | EP 2124748 | 08709966.9/2124748 | 200300 | 2009-548789 |
MX/a/2009/008469 | 578560 | 2,009,134,887 | 2009/05652 | 2359364 | 8,461,996


CONTENTS

1. IMPORTANT – READ FIRST!	2
2. Introduction	4
3. Parts Identification	5
4. Quick Start Guide	6
5. Diaper (Nappy) Fitment	6
6. Fitting your Go!	7
7. Operation	7
7.1 Startup	7
7.2 Monitoring	8
7.3 Travelling	9
7.4 Clothing	9
7.5 Baby Sound and Video Monitors	9
8. Settings	9
8.1 Audible Tick	9
8.2 Alarm Activation Period	10
8.3 Movement-Rate Warning	10
9. Battery	11
9.1 Battery Indicator	11
9.2 Battery Life	11
9.3 Replacing the Battery	12
9.4 Disposal	13
10. Maintenance	13
10.1 Precautions	13
10.2 Cleaning and Storage	13
11. Troubleshooting	14
12. Disclaimer	14
13. Guarantee	14

1. IMPORTANT – READ FIRST!

It is essential that Go!'s coloured flexible tip maintains constant contact with baby's tummy.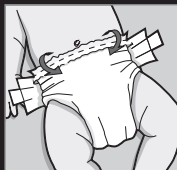

ALWAYS:


Use the correct size diaper (nappy). If it is too loose, Go! may lose contact.

Fold the top of the diaper over before attaching Go!, to ensure a firm grip. Use the nappy adjusters to provide a snug fit around the tummy.


Firmly clip Go! onto the folded nappy waistband, using the full extent of the clip.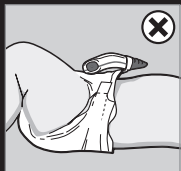



Ensure that Go! sits squarely on the nappy, and that the Movement Sensor (1) is in contact with baby's tummy.

Press the ON/OFF switch (5) to start monitoring.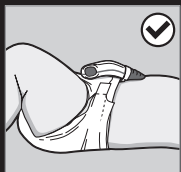

Always follow the above procedure to ensure that your baby's abdominal movement is being monitored, and to prevent false alarms caused by lack of contact with the baby's tummy.

NOTE: Handle Go! with care to prevent damage to clip or sensor.

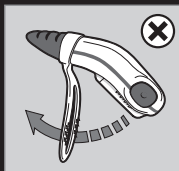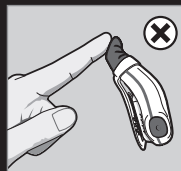2. INTRODUCTION

Snuza Go! is a portable baby movement monitor which safely and conveniently monitors your baby's abdominal movements while sleeping.

Go! can be clipped onto your baby's diaper (nappy) for total mobility. No cords, wires, sensor pads or external power are required.

Clip on, switch on and relax.

Go! detects even the slightest abdominal movement. If no abdominal movement at all is detected for a period of 20 seconds, Go! will emit an audible alarm to alert you. The 20-second period can be reduced to 15 or 18 seconds if preferred (see Section 8.2 Alarm Activation Period).

Go! will also alert you if your baby's abdominal movements are very weak or fall to less than 8 movements per minute.

**** Snuza Go! is not suitable for co-sleeping babies or bed sharing.**


3. PARTS IDENTIFICATION

Front

ENGLISH

Reverse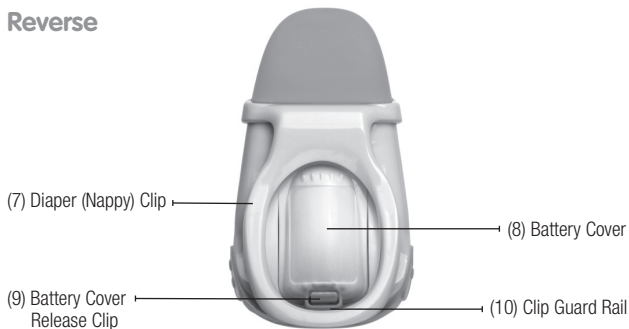


4. QUICK START GUIDE

1. **Important:** Use the correct diaper size and make sure it fits snugly (see Section 5 *Diaper/Nappy Fitment*).
2. Switch on by briefly pressing the ON/OFF button (5).
3. A start-up test will illuminate the indicator lights, play the start-up tone and perform a battery test.
4. Clip Snuzza Go! onto the diaper so that the Movement Sensor (1) is in direct contact with the baby's skin in the tummy area (see Section 6 *Fitting your Go!*).
5. The Movement Indicator Light (3) flashes on each abdominal movement. See Section 7.2 *Monitoring* for a full description of what it signifies.
6. If no abdominal movement is detected for a period of 20 seconds, the full audible alarm will sound and the Movement Indicator Light (3) will flash red. See Section 8.2 *Alarm Activation Period* for a description of how to reduce the time period to 15 or 18 seconds.
7. The alarm can be deactivated by briefly pressing the MODE/OFF button (6) and monitoring will continue.
8. To turn off the monitor, press both the ON/OFF (5) and MODE/OFF (6) buttons simultaneously. A shutdown tone will be heard.

5. DIAPER/NAPPY FITMENT

Important: Correct diaper fitment is essential for the operation of Go!. The thin flap of material at the top of the diaper must be folded over the thicker part before the diaper is closed (this is the section to which Snuzza Go! will be attached). The diaper should fit snugly on the baby with the top of the diaper just below the belly button. Loose fitting diapers will gape, causing the monitor to lose contact with the baby and false alarm. A very 'full' or wet diaper may also cause it to gape.

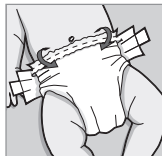

6. FITTING YOUR GO!

Ensure that the Battery Cover (8) is secure. Clip your Go! *all the way* onto the diaper to the full extent of the clip, as shown:-

Make sure that the Movement Sensor (1) is in contact with the baby's tummy around the belly button area, although closer to the sides may suit different sleeping positions.

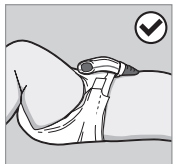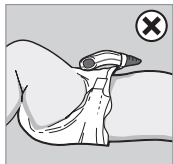The World Health Organisation (WHO) recommends that babies be placed to sleep on their back unless advised otherwise by a doctor.

7. OPERATION

7.1 Startup

Switch on by briefly pressing the ON/OFF button (5).

A startup-test will run which will flash all indicator lights and briefly sound the audible alarm.

The Battery Indicator Light (2) will flash briefly as follows:-

-  **Green** Battery OK
-  **Orange** Battery nearing depletion – **replace battery**
-  **Red** Battery flat – Go! will not monitor, **replace battery**

Warning: Ensure that indicator lights and alarm are working properly and that the battery level is high enough for proper operation (i.e. green or orange light). **Otherwise, DO NOT use the monitor.** If the battery is too flat to operate, the Battery Indicator Light (2) will flash red accompanied by a short

beep every second. **Change the battery before further use** (see Section 9.3 *Replacing the Battery*).

To switch OFF, press both the ON/OFF button (5) and the MODE/OFF button (6) simultaneously.

7.2 Monitoring

The Movement Indicator Light (3) flashes with each abdominal movement during monitoring:-

- | | |
|---|--|
|  Green | Normal abdominal movements. |
|  Orange | Very small or weak abdominal movements (eg very shallow breathing). |
|  Orange and
Movement-rate
Warning sounds
(dual-tone beep
every second) | Movement rate below 8 movements per minute
If the abdominal movement rate increases to
over 8 abdominal movements per minute, the
Movement Indicator Light will reset to green
(normal) and the warning will stop. To disable
this function, see Section 8.3 <i>Movement-Rate
Warning</i> . |
|  Red and full
audible alarm sounds | No abdominal movement at all has been
detected for the selected time period (15, 18 or
20 seconds). You can select the alarm
activation period (ie the period of no abdominal
movement detection) to be 15, 18 or 20
seconds (see Section 8.2 <i>Alarm Activation
Period</i>). The alarm can be deactivated by briefly
pressing the MODE/OFF button (6) or by
turning Snuza Go! off completely (press ON/
OFF (5) and MODE/OFF (6) buttons simul-
taneously). If the MODE/OFF button is used to
deactivate the alarm, monitoring will continue. |

7.3 Travelling

Movement from external sources such as being pushed in a pram or traveling in a vehicle or aircraft may be detected by Go!'s sensor. *Snuza Go! should not be relied upon where external sources of movement are present.*

7.4 Clothing

- Go! can be clipped onto the folded waistband of the diaper or the waistband of elasticated pants, as long as the Movement Sensor (1) touches the baby in the tummy area.
- There may also be a thin layer of clothing (eg a vest) between the Sensor and the skin, but ensure that the Sensor maintains contact.
- The monitor can also be used if baby is dressed in light clothing (for example, a Babygro) over the diaper or pants to which the Go! is attached.
- Too much clothing may attenuate or muffle the alarm. Make sure the alarm can be heard if baby is covered in clothing and/or blankets.

7.5 Baby Sound and Video Monitors (Optional)

Snuza Go! can be used together with a baby sound or video monitor to extend the distance at which the alarm can be heard.

8. SETTINGS

8.1 Audible Tick

This feature causes Go! to emit an audible 'tick' on each abdominal movement. To enable the Audible tick, ensure that Go! is switched ON, and press the MODE/OFF button (6) briefly (less than 1 second). To increase the volume of the tick, press the MODE/OFF button once again for medium volume and again for high volume. To turn off, briefly press the MODE/OFF button a fourth time – Go! will give a louder beep.

This feature may be used with a baby sound monitor to enable you to 'hear' baby's movements from a distance.

8.2 Alarm Activation Period

Decide whether you prefer the alarm to activate after 15, 18 or 20 seconds of no abdominal movement.

Ensure that Snuzo Go! is switched OFF.

Press and hold the MODE/OFF button (6) for 5 seconds until the unit beeps and both indicator lights flash briefly.

The Movement Indicator Light (3) will then start flashing with a pause between flashes. The following number of flashes relate to the alarm activation period:-

- 1 flash – 15 seconds
- 2 flashes – 18 seconds
- 3 flashes – 20 seconds

The factory pre-set value is for 20 seconds (3 flashes).

Each press of the ON/OFF button (5) will increase the number of flashes from 1 to 3. Press the ON/OFF button until the desired number of flashes (relating to the desired setting) is reached. Press the MODE/OFF button (6) to store the setting. Go! will be in the power OFF mode after the setting has been saved. If the settings mode is entered and no button presses are detected within 30 seconds, Go! will switch to the monitoring mode (as if the unit had been turned on using the ON/OFF button).

8.3 Movement-Rate Warning

The Movement-rate Warning alerts you if the baby moves less than 8 times per minute – the Movement Indicator Light (3) flashes orange and the Movement-rate Warning sounds (dual-tone beep every second). This feature is automatically enabled each time Go! is switched ON.

To disable this warning during monitoring, ensure that Go! is switched ON, and press and hold the MODE/OFF button (6) for more than 1 second until the unit beeps.

If you want to enable the warning again, switch Go! off completely (press both ON/OFF (5) and MODE/OFF (6) buttons simultaneously), then switch it ON again. The Movement-rate Warning is enabled by default when Go! is switched on.

9. BATTERY

9.1 Battery Indicator

If the battery becomes depleted during monitoring, the Battery Indicator Light (2) will flash as follows:-

- | | | |
|---|--|--|
| ● | Red flash on each abdominal movement | Nearing depletion – replace battery before further use. |
| ● | Red flash on each abdominal movement and the low battery alarm beeps every 2 seconds | Battery flat – no monitoring will take place. Replace battery before further use. |

If the battery is nearing depletion or depleted, change the battery before further use of the monitor (see Section 9.3 *Replacing the Battery*).

9.2 Battery Life

The life of the replaceable battery depends entirely on usage. Battery life will vary significantly depending on the average daily duration of use and/or alarm duration and/or number of alarms. Turning on and off unnecessarily shortens the battery life. If the number of alarms is high, check diaper fitment. Please note that the product guarantee does not apply to the battery.



9.3 Replacing the Battery

For all SE models: use a CR2 (CR15270) battery available from most retail outlets.

For all other models: use only a 14250 battery that delivers 3V 600mAh, available from a specialist battery supplier or your national Snuzo distributor. Please see our website www.snuzo.com for information about batteries and details of your national distributor.

*** The serial number is located inside the battery housing (it can also be found on the original Snuzo packaging). To locate the serial number, remove the battery cover and battery as per the instructions below:*

1. Use your fingernail or a small flat implement. With a pinching action (Figure 1), gently pull back the Battery Cover Release Clip (9) on the underside of the monitor and lift the Battery Cover at the same time (Figure 2).

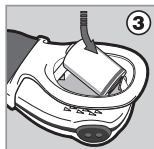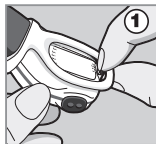2. The Clip Guard Rail (10) prevents the clip from bending too far. Do not try to bend the clip beyond this point.

3. Remove the depleted battery. *The serial number is located beneath the battery.*

4. Insert the new battery, negative end first, as shown in Figure 3. Ensure that the polarity is correct (align the positive end '+' of the battery with the '+' symbol in the battery housing). Replace the Battery Cover and ensure that it is correctly closed (Figure 4).

5. Test the unit by switching on and ensuring that the Battery Indicator Light (2) flashes green.

6. If the unit starts emitting a beep after the battery has been replaced, simply press the ON/OFF (5) and MODE/OFF (6) buttons simultaneously to turn it off.


9.4 Disposal

The battery must be disposed of in accordance with your country's laws governing the disposal and recycling of batteries.

10. MAINTENANCE

10.1 Precautions

- Do not insert anything into the Speaker Holes (4).
- Do not allow any liquid to enter the Speaker Holes.
- Avoid bending the sensor unnecessarily (fig 1).
- Do not use undue force to bend the clip (fig 2).
- Do not pull the Battery Cover Release Clip (9) beyond the Guard Rail (10).

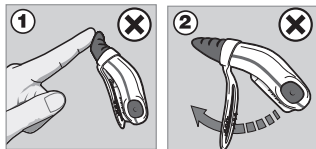10.2 Cleaning and Storage

- Gently wipe with a damp cloth to remove any dirt. A mild soap may be used for stubborn dirt.
- Do not use any other cleaning agents as they may contain damaging chemicals.
- The monitor is splash resistant, not waterproof. Do not immerse in water. If it gets wet, dry it off immediately.
- When changing the battery, dirt that is trapped between the battery replacement hatch and the case may be cleaned with a dry cotton bud once the Battery Cover (8) has been removed. Be sure that the rubber seal is free of dirt before replacing the Battery Cover, as this could compromise the seal.
- Do not store in a damp area.

11. TROUBLESHOOTING

1. Check incorrect fitment and placing of Go! on baby's diaper (see Section 5 *Diaper Fitment* and Section 6 *Fitting your Go!*).
2. Battery failure: The Battery Indicator Light (2) will show red (see Section 9.3 *Replacing the Battery*).

Please visit our website www.snuza.com for further information.

12. DISCLAIMER

Customers use this monitor at their own risk and Biosentronics cc, its members, employees, agents, contractors, representatives, manufacturers, distributors and associates ("SNUZA") shall not be held liable for any loss, damage or injury suffered by the customer or any other person arising from, or in connection with, the use or manufacture of the monitor, including loss, damage or injury suffered by the customer or any other person as a result of any defects in respect of the monitor, and all warranties, and any other liability from whatsoever cause arising, are hereby excluded and the customer hereby indemnifies Biosentronics cc in respect of any such liability.

13. GUARANTEE

Biosentronics cc guarantees this product to be free from defects in materials and workmanship for a period of 12 months from the original date of purchase, verified by a sales receipt.

Our guarantee liability shall be limited to replacing the unit free of charge and does not include transport costs. Our guarantee extends to all countries in which this product is supplied. By the act of operating this product, the user accepts all resulting liability. We reserve the right to modify the provisions of this guarantee at any time without notice. This guarantee does not cover the battery, incorrect application, tampering, misuse or transport damage.

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