snuza[®] **HERO^{SE}**

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Thank you
for choosing
snuza[®]

and for letting us be a part of your
parenting journey.

We would love to see a photo of your little
one wearing their Snuzza monitor!

Please tag us on social media:

@snuzamonitors

@SnuzaUSA

We'd greatly appreciate your review on our
social media platforms, Amazon or via
support@snuza.com.

Your feedback helps to further our work in
promoting safe sleep for infants worldwide.

IMPORTANT: Register your Snuzza monitor to
validate its authenticity. Follow the instructions in
the Warranty section of this booklet.

The Snuzza Family
www.snuza.com

30 Days extended warranty

IMPORTANT: proof of purchase is required to validate warranty.

This monitor comes with a 1-year warranty that starts from the date of original purchase, verified by a sales receipt. We'll extend your warranty by 30 days when you register your device within 7 days of purchase.

Here's how to extend your warranty:

1. A. Register as a user on snuza.com/register, follow the prompts to complete registration and add your device, **OR**
B. If you have already registered, log in and scan the QR code below to register your device, or navigate to www.snuza.com/account/device/add/.
2. Enter your device's serial number and your child's details.
3. Upload your proof of purchase and enter the purchase date, and save.

To get product support:

1. Log into snuza.com.
2. Go to your product's dashboard and click on your device's serial number.
3. Click on the 'Get Support' button.
4. Enter your product support query, then click the 'Send' button.
5. We will be in contact via email to try and resolve your issue.
6. If we are unable to resolve the issue, then the warranty claim process becomes available.
7. You can do this either on your product's dashboard by clicking the 'Report fault' button or the link provided in the email that you will receive.

If you have any feedback or questions, please contact us via your product's dashboard by clicking the 'Send a message' button or email us at support@snuza.com.

Welcome to the Snuza Family!
The Snuza Team

Hero^{SE} Quick Start Guide

Scan the QR code for more detailed information

For support contact us through support@snuza.com

1. Prepare

- 1.1 Switch on by pressing the 'ON/OFF' button.
- 1.2 A start-up test will illuminate the Indicator lights, test the Pre-Alert Vibration function, play the start-up tone and perform a battery test. Your Hero^{SE} is now ready for use.
- 1.3 **IMPORTANT:** Handle the Hero^{SE} carefully to avoid damaging the clip or sensor. Do not bend the Movement sensor.

2. Attach

- 2.1 Fold over the top edge of the nappy/diaper.
- 2.2 Attach the Hero^{SE} squarely onto the folded edge of the nappy/diaper.
- 2.3 Ensure the Hero^{SE} fits snugly and the Movement sensor is in contact with the baby's tummy.
- 2.4 **IMPORTANT:** Ensure no skin pinching by gently sweeping your fingers between the skin and clip.

3. Monitor

- 3.1 When movement is detected, the Movement Indicator Light flashes with each movement.
- 3.2 If no movement is detected for 15 seconds, the Hero^{SE} will activate a Pre-Alert Notification with Vibration and beep with a red flashing Movement Indicator Light.
- 3.3 If movement is detected within 5 seconds after the Pre-Alert Notification with Vibration, the Movement Indicator Light flashes red on each detected movement as a Pre-Alert Indication to indicate that there has been a 15 second period where no movement was detected.

- 3.4 After three Pre-Alert Notification with Vibration occurrences, the Repeated Pre-Alert Notification will be activated (red flash and beeps every 2 seconds). This feature can be deactivated, see 8.2 of the user manual.
 - 3.5 If the period of no movement detected reaches 20 seconds, an audible alarm will sound and the Alarm Indicator Light (4) will flash red. The caregiver must check the reason for no movement detected.
 - 3.6 The Slow Movement Notification alerts you if detected movement rate slows down. It is identified by the Movement Indicator Light (3) flashing yellow with a simultaneous dual-tone audible beep every second. This feature is automatically enabled each time Hero^{SE} is switched ON, but it can be deactivated. See section 8.3 of the user manual for more details.
 - 3.7 To turn off, press the 'ON/OFF' and 'MODE/OFF' buttons simultaneously.
-

Precautions:

- To avoid pinching the baby's skin, gently sweep your fingers between the clip and skin after attachment.
 - Only use the Hero^{SE} when the baby is in a motionless bed/cot/crib.
 - Not suitable for co-sleeping or bed sharing.
 - Movement from external sources such as being pushed in a pram/stroller or travelling in a vehicle or aircraft may be detected by Hero^{SE}'s sensor.
 - Hero^{SE} should not be relied upon where external sources of movement are present.
 - This movement monitor may not be able to detect life threatening events.
-

DISCLAIMER: ALL SNUZA PRODUCTS ARE SUPPLIED STRICTLY SUBJECT TO THE TERMS AND CONDITIONS CONTAINED IN THEIR USER MANUALS AND PUBLISHED BY SNUZA INTERNATIONAL (PTY) LTD AND ITS AFFILIATES ("SNUZA") FROM TIME TO TIME. THIS INCLUDES SNUZA PRODUCTS THAT ARE SUPPLIED THROUGH THIRD PARTY DISTRIBUTORS, RETAILERS AND RESELLERS. IN PURCHASING AND/OR USING A SNUZA PRODUCT, CONSUMERS EXPLICITLY ACCEPT THESE TERMS AND CONDITIONS AND ACKNOWLEDGE THAT THEIR USE OF SNUZA PRODUCTS IS ENTIRELY AT THEIR OWN RISK. CONSUMERS ARE SOLELY RESPONSIBLE FOR ENSURING THAT SUCH USE IS IN ACCORDANCE WITH THE DIRECTIONS PUBLISHED BY SNUZA FROM TIME TO TIME.

DISCLAIMER: TO THE MAXIMUM EXTENT PERMISSIBLE BY LAW, SNUZA EXPRESSLY DISCLAIMS AND EXCLUDES ANY AND ALL LIABILITY AND RESPONSIBILITY OF WHATSOEVER NATURE AND HOWSOEVER ARISING (WHETHER IN CONTRACT, DELICT, TORT, NEGLIGENCE, WARRANTY, STRICT LIABILITY OR OTHERWISE) FOR ANY DAMAGE, INJURY, DEATH, HARM, COST, EXPENSE OR LIABILITY OF WHATEVER NATURE WHICH MAY BE INCURRED OR SUSTAINED BY ANY PERSON ARISING OUT OF OR IN CONNECTION WITH THE USE OR MISUSE OF ANY SNUZA PRODUCT, INCLUDING DEFECTIVE PRODUCTS. SNUZA PROVIDES A LIMITED GUARANTEE AND WARRANTY ON CERTAIN OF ITS PRODUCTS. TO THE MAXIMUM EXTENT PERMISSIBLE BY LAW, SNUZA EXPRESSLY DISCLAIMS ALL OTHER WARRANTIES OF ANY KIND, WHETHER EXPRESS, IMPLIED, STATUTORY OR OTHERWISE, AND INCLUDING (WITHOUT LIMITATION) WARRANTIES OF MERCHANTABILITY, FITNESS FOR PURPOSE, TITLE, INFRINGEMENT OF THIRD PARTY RIGHTS, AND QUALITY IN RESPECT OF ITS PRODUCTS. IN NO EVENT SHALL SNUZA OR ITS DIRECTORS, OFFICERS, EMPLOYEES, AGENTS, MANUFACTURERS, DISTRIBUTORS, SUPPLIERS AND AFFILIATES BE HELD LIABLE FOR ANY DEFECTS OR DAMAGE TO ANY SNUZA PRODUCT EXCEPT TO THE EXTENT COVERED BY AN EXPRESS GUARANTEE OR WARRANTY PROVIDED BY SNUZA. THE CONSUMER INDEMNIFIES SNUZA AND ITS DIRECTORS, OFFICERS, EMPLOYEES, AGENTS, MANUFACTURERS, DISTRIBUTORS, SUPPLIERS AND AFFILIATES AND HOLDS THEM HARMLESS FROM AND AGAINST ANY CLAIMS, DAMAGE, LOSS, HARM, LIABILITY, INJURY OR DEATH WHICH MAY DIRECTLY OR INDIRECTLY RESULT FROM THE USE, MISUSE, DEFECT OR DAMAGE TO ANY SNUZA PRODUCTS BY ANY PERSON. **GUARANTEE:** SNUZA INTERNATIONAL (PTY) LTD GUARANTEES THIS PRODUCT TO BE FREE FROM DEFECTS IN MATERIALS AND WORKMANSHIP FOR A PERIOD OF 12 MONTHS FROM THE ORIGINAL DATE OF PURCHASE, VERIFIED BY A SALES RECEIPT. OUR GUARANTEE LIABILITY SHALL BE LIMITED TO REPLACING THE UNIT FREE OF CHARGE AND DOES NOT INCLUDE TRANSPORT COSTS. OUR GUARANTEE EXTENDS TO ALL COUNTRIES IN WHICH THIS PRODUCT IS SUPPLIED. BY THE ACT OF OPERATING THIS PRODUCT, THE USER ACCEPTS ALL RESULTING LIABILITY. WE RESERVE THE RIGHT TO MODIFY THE PROVISIONS OF THIS GUARANTEE AT ANY TIME WITHOUT NOTICE. THIS GUARANTEE DOES NOT COVER THE BATTERY, INCORRECT APPLICATION, TAMPERING, MISUSE OR TRANSPORT DAMAGE.

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